



Prepare your test area

TIP

Do not eat, drink, smoke or vape 30 minutes prior to or during the test.

ADVICE

You'll need a timer (a watch or clock) during your test.

- (i) Read instruction guide carefully enclosed in box by manufacturer.

You can also watch the **Swab Test Video Online** if you need more help:

- <https://youtu.be/dk3LLAD7ybk>

<https://www.gov.uk/guidance/covid-19-self-test-help>

- (ii) Clear, clean and dry a flat surface immediately before starting the test.
- (iii) Wash or sanitise your hands. Make sure they are dry before starting the test.

Check your kit contents

Rapid Antigen Test Kit Contents

Extraction Tube with Liquid & Tube Tip in Plastic Bag	x 1	Rapid Antigen Test Result Reporting Form	x 1
Test Device in sealed pack	x 1	Instruction & Procedure sheet	x1
Sealed Disposable Sterile Swab	x 1		

ADVICE

What to do if something is damaged, broken or missing if you notice anything damaged, broken, missing or difficult to use in the kit, do not use it. We will send you replacement immediately. You can report to our customer services team.

Set up your test.

Prepare yourself & follow the instruction sheet provided in test kit.

Cautions:

- Once opened, start the test within **30 minutes**.
- If the liquid in the extraction buffer tube gets onto the skin or into eyes, wash immediately and seek medical help if necessary.
- Avoid touching the fabric tip of Swab.

Take your swab sample:


Follow the instruction sheet provided in test kit.



Important Stop Here to take Photo:

1st Photographic Evidence:

Please take Picture of your Face holding Swab in your nostril (2.5cm inside)

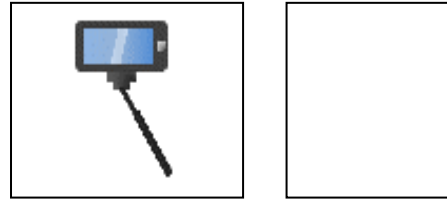

Process the swab sample & Analyse Result

Follow the instruction sheet provided in test kit.

Fill the “Rapid Antigen Test Result Reporting Form:

Once Test procedure completed & you have seen the clear result on testing device.

- Fill the “Rapid Antigen Test Result Reporting Form” & sign it.
- Place the “Test device” on “Rapid Antigen Test Result Reporting Form” provided in the kit as shown in figure.

Important Stop Here to take Photo:

2nd Photographic Evidence:

Please take a clear Picture of Test Result Reporting form. We require high quality of picture to read result clearly.



Register your kit Online. – Final Step

- ✓ Register your Kit online <https://medplustest.uk/register-rapid-antigen-test/>
- ✓ Upload following two Photographic Evidence:
 - 1st Photo Graphic Evidence of your face with SWAB in your nostril.
 - 2nd Photo of Test Device on Rapid Antigen Result Reporting Form.
- ✓ Passport Copy or Photo ID to identify your identity.

Cautions: Please make sure you upload images very clear & readable. We will not be able to process "Test Certificate" if uploaded images are blurry or not displayed full. We will not be able to issue Result Certificate without these evidences.

If you have any difficulty to upload your files or our server down. Please contact us

Result Notification:

Your result certificate will be sent within 2-3 hrs by e-mail after your Kit Registration completed subject to if you have provided all corre

Customer Services & Help Centre

medplus Test
www.medplustest.uk
results@medplustest.uk

Tel: 0333 433 0223
WhatsApp: 07888 040407
Hotlines: 07999 905185